




# Access Health

# EquaGel - Waterproof Continence Cover

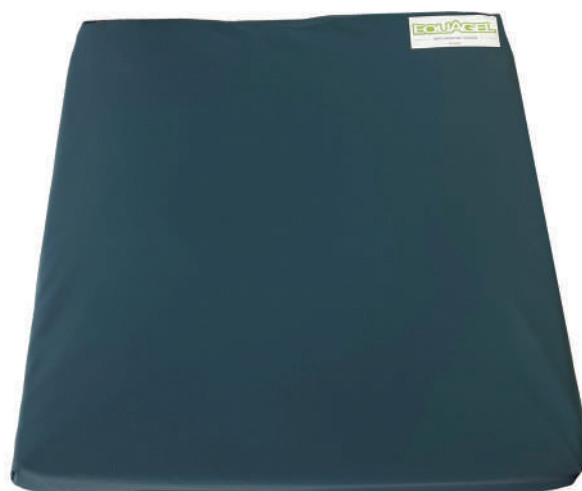

## EQUAGEL WATERPROOF COVER

The EquaGel Waterproof Cover is a hygienic and super comfortable, waterproof cover suitable for use within hospitals and aged care facilities.

This cover provides the perfect solution to cleaning and hygiene requirements.

## KEY FEATURES

- Antibacterial, antifungal, waterproof, breathable and flame retardant
- Easy clean - soapy water or diluted disinfectant wipe down
- Fully zippered enclosures to make fitting easy and provide complete protection
- Waterproof infection control zippers that can be wiped down with diluted disinfectants

## SIZE RANGE

Available in two sizes to fit complete EquaGel cushion range

### CE2470 EquaGel Waterproof Cover - Regular (W 46 x D 46 x H 6.5cm)

Suitable for:

#### General

CE2004 40x40cm (16"W x 16"D)  
CE2006 40x46cm (16"W x 18"D)  
CE2010 46x40cm (18"W x 16"D)  
CE2012 46x46cm (18"W x 18"D)

#### Protector

CE2104 40x40cm (16"W x 16"D)  
CE2106 40x46cm (16"W x 18"D)  
CE2108 46x40cm (18"W x 16"D)  
CE2110 46x46cm (18"W x 18"D)

#### Adjustable

CE2204 40x40cm (16"W x 16"D)  
CE2206 40x46cm (16"W x 18"D)  
CE2208 46x40cm (18"W x 16"D)  
CE2210 46x46cm (18"W x 18"D)



### CE2471 EquaGel Waterproof Cover - Large (W 56 x D 46 x H 6.5cm)

Suitable for:

#### General

CE2008 40x51cm (16"W x 20"D)  
CE2014 46x51cm (18"W x 20"D)  
CE2016 51x40cm (20"W x 16"D)  
CE2018 51x46cm (20"W x 18"D)  
CE2020 51x51cm (20"W x 20"D)  
CE2022 56x41cm (22"W x 16"D)  
CE2024 56x48cm (22"W x 18"D)

#### Protector

CE2112 51x40cm (20"W x 16"D)  
CE2114 51x46cm (20"W x 18"D)  
CE2116 56x40cm (22"W x 16"D)  
CE2118 56x46cm (22"W x 18"D)

#### Adjustable

CE2212 51x40cm (20"W x 16"D)  
CE2214 51x46cm (20"W x 18"D)  
CE2216 56x46cm (22"W x 16"D)  
CE2218 56x46cm (22"W x 18"D)

Ph: 03 9878 1399

| sales@accesshealth.com.au

| www.accesshealth.com.au





Access Health

# EquaGel - Waterproof Continence Cover



## CARE OF THE EQUAGEL WATERPROOF COVER

### WASHING AND DISINFECTION

- EquaGel Waterproof Covers need to be treated with care when cleaning the surface. Many industrial cleaning agents are extremely harsh on the coating and can cause it to break down.
- The normal cleaning instructions are to wipe the surface clean with a sponge or cloth with soapy water.
- EquaGel Waterproof Covers can be machine washed in soapy water at up to 95°C.
- Do not use solvent on EquaGel Waterproof Covers.
- EquaGel Waterproof Covers are able to be autoclaved at a temperature of 134°C for 3 minutes. However, care must be taken to first clean the surface with pure water and then to make sure the EquaGel Waterproof Covers are not stacked on top of each other without any air gap separation.
- Some surface wrinkling may result from cleaning procedures.
- Abrasive cleaning agents should not be used.
- EquaGel Waterproof Covers are dry-cleanable with all dry cleaning fluids except trichloroethylene.
- To avoid premature degradation of the surface coating never use an abrasive cloth, sponge or other similar device on the surface. Carefully follow manufacturer instructions for recommended exposure time of cleaner on surface. Wipe off cleaning product residue with a clean, soft cloth after cleaning and allow the surface to dry completely before use.

### DRYING

- EquaGel Waterproof Covers can be tumble dried provided the operator ensures it is removed as soon as all moisture is removed.
- EquaGel Waterproof Covers **MUST NOT** be left on the heated bowl of the dryer when drying is complete.
- **DO NOT** mangle.

### IRONING

- **DO NOT** iron EquaGel Waterproof Covers.

### STORAGE

- Store in a cool dry area. Avoid excessive pressure and contact with non-colourfast material.

\*CAUTION: Please ensure cushion is removed prior to cleaning.



# Access Health

## EquaGel - Waterproof Continance Cover



### APPROVED CLEANERS

MANUFACTURER	PRODUCT	MAIN ACTIVE COMPONENT	CONCENTRATION	DATE TESTED
Ecolab®	A-456-II		Minimum 256 to 1 dilution with water	02-Dec-14
Quantum Pacific Limited	Actimax		According to Manufacturer Spec	11-Apr-06
SCIENCECompany	Ammonium Chloride	Cocoalkyldimethylbenzyl ammonium chloride	According to Manufacturer Spec	17-May-19
Milton®	Anti-Bacterial Tablets	Sodium Dichloroisochyanurate	1 Tablet: 2L Water	19-Jul-12
Ecolab®	Asepti Active Liquid		According to Manufacturer Spec	20-Nov-12
Synergy Health	Azomax™ Multi Surface Wipes	Benzylkoniumchloride 21.6%	Ready to Use	15-Mar-13
PreVasive®	BAC Botanical Antimicrobial Cleaner	Thyme Oil	According to Manufacturer Spec	22-May-19
Spartan Chemical Company Inc.	BNC-15™		According to Manufacturer Spec	11-Aug-16
Metrex® Research	CaviCide®	Isopropanol	According to Manufacturer Spec	06-Oct-16
Metrex® Research	CaviCide1™	Isopropanol	According to Manufacturer Spec	05-Jan-17
Agar Cleaning Systems Pty. Ltd.	Chloradet	Sodium Hypochlorite <10%	Minimum 1000 to 1 dilution with water	
Clorox Disinfecting Wipes	Clorox Disinfecting Wipes		According to Manufacturer Spec	14-Feb-11
Clinell®	Detergent Wipes	Propane 1,2-diol	According to Manufacturer Spec	27-May-09
Clorox®	Dispatch® Disinfectant		According to Manufacturer Spec	16-Feb-12
Clorox HEALTHCARE®	Fuzion™		According to Manufacturer Spec	05-Jan-17
3M	HB Quat Disinfectant	Alkyl dimethyl benzyl ammonium chloride	Ready to Use	06-Sep-19
Wipesplus	Hospital/surgical wipes	Ammonium Chloride	Ready to Use	15-May-19
Medline Industries Inc.	MICRO-KILL™ AF2 Wipes	Ammonium Chloride	Ready to Use	15-May-19

Ph: 03 9878 1399

| sales@accesshealth.com.au

| www.accesshealth.com.au

